# The Stork® OTC Conception Device

- Available without a prescription
- Assist conception in the privacy of home
- **Cost-effective:** a cost alternative to more invasive, costly, in-clinic treatments
- Easy to use: 3 simple steps

# **Testimonials**



"My wife and I got married while I was in the Army and between my deployments and careers we ended up waiting until our we were almost 40 to try to start a family. We both had a lot of concerns about our ability to conceive but fortunately the Stork was available. We were able to use it in the privacy of our own home and after only two tries, we conceived. We're 6 months along now and we couldn't be happier!"

- A.K.Z.. US

"Thank you!!! After over a year of trying to conceive and finding out husband had a few sperm related issues, we got pregnant on our secon month using the stork!... we are thrilled and thankful that The Stork is available and I will recommend it to others."

Sarah L.

"My husband and I have been trying to conceive for nearly two years He has a low sperm count which has contributed to the problem. The month we used The Stork along with Premama Fertility Drink and FertiliTea and at 11 DPO we have tested positive... I recommend The Stork. I don't think I'd have that HPT Positive without it."

- Johnnae W., US

"The Stork OTC is an innovative tool that will certainly be of interest t patients...The Stork OTC has the advantage of being able to be used o home. Thus, patients do not have to have the inconvenience of being seen in the doctor's office."

- Dr. James M. Goldfarb, MD MBA

### Where to Purchase

For information on where to purchase, please visit our website at www.storkotc.com and click "Shop Now" for a full list of retailers. You can also purchase direct from our website.

Available online and in-store at select retailers:







Walgreens com

Complete Instructions for Use are provided with The Stork OTC device. Please read these carefully prior to use. For a video version of the instructions, please visit our website at: www.storkotc.com.

#### **Questions?**

Always speak to your Healthcare Professional if you have any concerns or questions about your fertility or health in general.

If you have any questions relating to The Stork OTC, please contact Customer Service at info@rinovum.com or by phone at 1.412.200.7996.

#### About Rinovum® Women's Health

Rinovum Women's Health is a privately held women's health company based in the U.S. dedicated to bringing products into the market that will enhance women's lives and empower them to take charge of their health. The Stork OTC is a device for reproductive health to aid in conception while in the privacy and comfort of home. We aim to provide products that are easy-to-use, safe, and offer an additional option before more expensive treatments

The Stork Ib2C, Inc.,
A Rinovum Women's Health Company
300 Oxford Drive, Suite 330, Monroeville, PA 15146
+1 (412) 200-7996

STOP Take a shortcut on the path to conception. www.StorkOTC.com

© 2016 The Stork Ib2C, Inc., A Rinovum Women's Health Company. 1008-475 R01

## The Stork® OTC

The Stork OTC is a uniquely designed medical device you use at home, during ovulation, to assist your chances of becoming pregnant. Our FDA-cleared, over-the-counter device combines an innovative applicator and an established conception technique – cervical cap insemination. This smart technique enables the sperm to guickly bypass the vaginal tract and places the sperm as close as possible to the opening of the cervix, right where it needs needs to swim up through to get to the egg.

The Stork OTC is designed to be used as a "next step" for you and your partner when trying to conceive, before advancing into more costly in-clinic reproductive treatments.

### How Does The Stork® OTC Work?


Its simple design includes two main parts – the Conceptacle® and the applicator. The Conceptacle consists of a latex-free, condom-like sheath with a cervical cap resting within the sheath. The Conceptacle is used to collect the semen containing sperm, worn during intercourse, or alternatively the device can be used with donor sperm. The cervical cap is then removed and placed into the applicator. The applicator cradles the cervical cap and guides it through the vagina to the cervix. This delivers the sperm next to the cervix. The cervical cap is then removed after 4-6 hours using a tampon-like pull (string), optimizing your chances of conceiving.

<sup>1</sup> Flierman, Hendrikus, et al. A Prospective, randomized, cross-over comparison of two methods of artificial an Reproduction, Vol. 12, no.9 1945-1948, 1997. 2 Bergquist, C.A., et al. Artificial insemination with fresh donor semen using the cervical cap technique: a review of 278 cases. Obstet Gynecol. 1982 Aug;60(2):195-9. 3 Corsono SL, Batzar FR, Otis C, Fee D. The cervical cap for home artificial insemination J. Reprod. Med., 1986, May; 31 (5): 349-52. 4 "A Prospective, randomized, cross-over comparison of two methods of artificial insemination by donor on the incidence of conception: Intracervical insemination by straw versus cervical cap", Human Reproduction, Vol. 12, no.9 1945-1948, 1997, Flierman, Hendrikus, et al. 5 "Evaluation of the effect of a cervical cap device on sperm functional characteristics in vitro". ANDROLOGIA 33, (2001), M. C. Mahony. \* "Postcoital Sperm Assessment Comparative Study." Michael. J. Pelekanos, MD. Surgical Technology International, XXVII, December 2015.



# **About Cervical Cap Insemination**

Up to 20% of women who use cervical cap insemination, the technique of the Stork OTC, get pregnant. Comparable to IUI with recorded success rates between 16-21%.\*

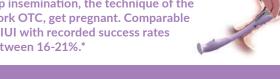

of subjects, The Stork OTC delivered 3.23 times the sperm intercourse and ... presents a greater concentration of sperm to targeted cervical mucus than natural intercourse.

# Frequently Asked Questions

#### When do I use The Stork OTC?

## Do I need a prescription for The Stork OTC?

